


Mobile phone sensor

Using mobile phone sensor with the alarm clock

- 1 Open the battery compartment, fit the supplied battery and connect the mobile phone sensor to the 2.5 mm input.
- 2 Connect the adapter to the 3.5 mm ext. trig. input on the back of the alarm clock.
- 3 Place the sensor on the mobile phone or tablet display and use e.g. a landline telephone to call the mobile phone. When the display lights up, the yellow Visit LED on the alarm clock blinks and it starts to sound, flash and vibrate.

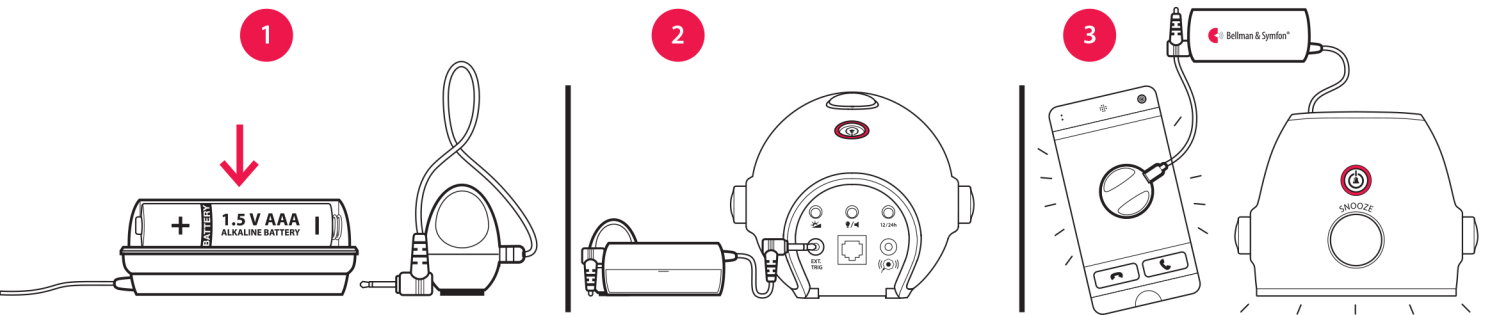

Using mobile phone sensor with the flash receiver

- 1 Connect the mobile phone sensor to the 2.5 mm ext. trig. input on the back of the receiver. Place it on the mobile phone or tablet display.
- 2 Use for instance the landline telephone to call the mobile phone. When the mobile phone display lights up, the yellow Visit LED on the receiver lights up and it starts to flash.



Using mobile phone sensor with the telephone transmitter

- 1 Open the telephone transmitter front cover and connect the mobile phone sensor to the 2.5 mm ext. trig. input. Place it on the mobile phone or tablet display.
- 2 Use for instance the landline telephone to call the mobile phone. When the mobile phone display lights up, the transmitter top LED lights up in green to show that a radio signal is being transmitted.
- 3 The yellow Visit LED on the receiver lights up to show that the signal was received. In addition it starts to sound, flash or vibrate depending on the receiver.

